

SELECT FLOWMETER



The Select Flowmeter allows you more treatment options from one gas source.

It's Versatile...

The Select Flowmeter allows you to direct the output of the flowmeter to any ONE of the three (3) outlet connections.

It Saves Time...

You can deliver a nebulizer treatment without disconnecting the humidifier.

It Eliminates Clutter...

The Select Flowmeter has multiple ports, you can leave the cannula, humidifier and nebulizer hooked up to the outlet for easy treatment

whenever needed. No more loose tubing lying around waiting to be used for treatments.

It's Easy to Use...

Simply push the button on the bottom of the unit and rotate the desired port to the large, "ON" arrow (▼) for treatment. This shuts off gas from one port and redirects it to the port to which another device is connected. When the optional treatment is finished, simply push the button on the bottom of the unit and rotate the desired port to the large "ON" arrow.


It's Durable...

The Select Flowmeter combines the durability of a metal flowmeter with its DISS male connection along with a tough nylon body.

They're Dependable...

The housing and flow tube carry a lifetime warranty and the needle valve carries a five (5) year warranty.



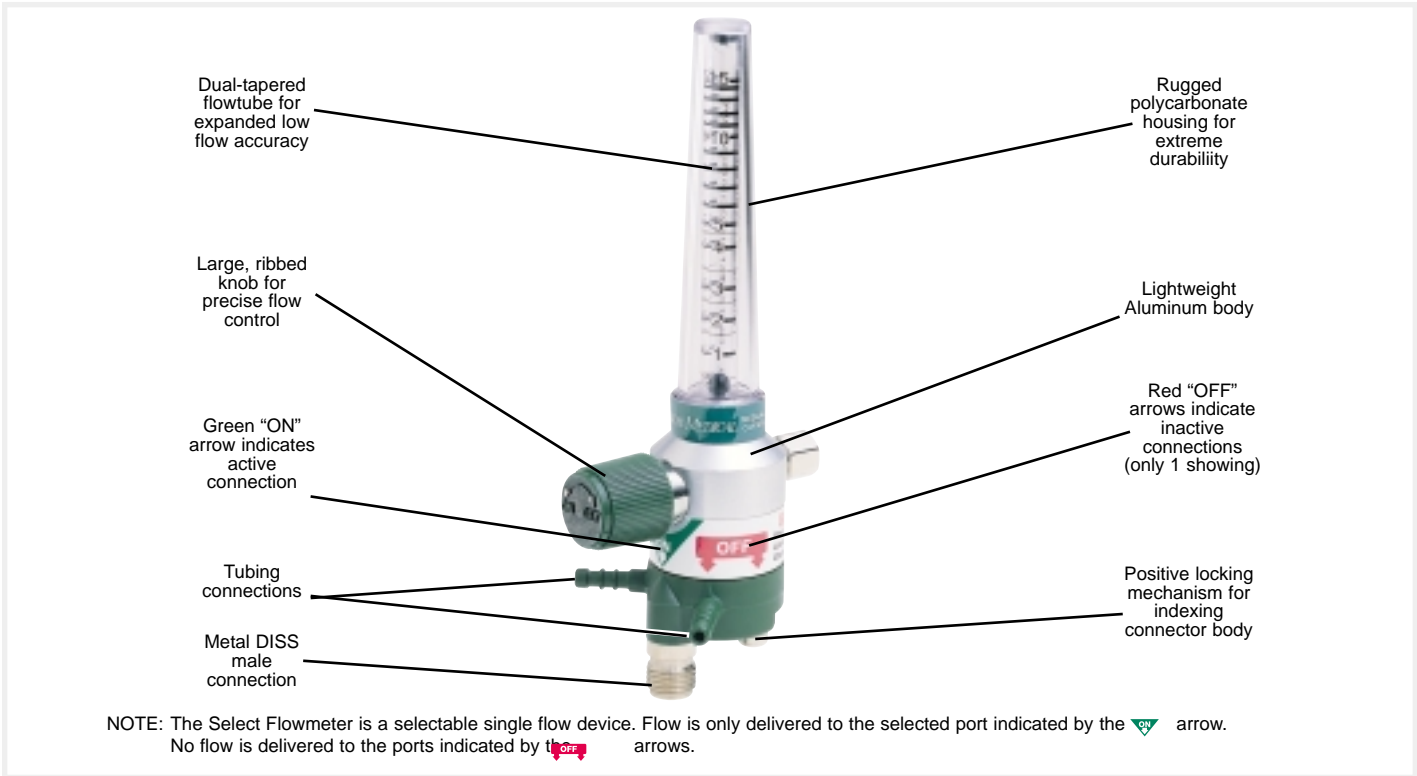

ORDERING INFORMATION (SELECT FLOWMETER)

	No Adaptor	3MFA1001
	DISS Female Hex Nut	3MFA1002
	DISS Female Hand Tight	3MFA1003
	DISS Male	3MFA1004
	Ohmeda	3MFA1005
	Chemetron	3MFA1006
	Oxequip	3MFA1007
	Puritan-Bennett	3MFA1008
	Schrader	3MFA1009

SPECIFICATIONS

For Power Take-Off, add PTO after part number.

Gas	Scale	Increments	Accuracy	Maximum Flush Flow
Oxygen	0-15 lpm	.5 lpm to 5 lpm	±.25 lpm from .5 to 5 lpm	70 lpm
		1 lpm from 6 to 15 lpm	±.5 lpm above 5 to 15 lpm	Min

Flowmeters calibrated at 50 PSI, 70°F, standard atmospheric pressure. Specifications are subject to change without prior notice.



Precision Medical, Inc.
 300 Held Drive • Northampton, PA 18067
 Phone: 610-262-6090 Toll-Free Phone: 1-800-272-7285
 Fax: 610-262-6080 Toll-Free Fax: 1-800-352-1240
 Visit us on the Web: www.precisionmedical.com