

## Troubleshooting .....

If problem cannot be corrected, consult your dealer/Homecare equipment provider.

PROBLEM	PROBABLE CAUSE	REMEDY
A. No flow	<ol style="list-style-type: none"><li>1. Cylinder valve closed</li><li>2. Conserver in "OFF" position</li><li>3. Cylinder empty</li><li>4. Dial not at setting</li><li>5. Conserver not sensing breath</li></ol>	<ol style="list-style-type: none"><li>1. Turn on cylinder</li><li>2. Set to prescribed setting</li><li>3. Replace cylinder</li><li>4. Set dial so indicator points to a setting</li><li>5. Check position of cannula in nose Do not breathe through mouth</li></ol>
B. Leaking at Yoke	<ol style="list-style-type: none"><li>1. Missing or defective Yoke Seal</li></ol>	<ol style="list-style-type: none"><li>1. Replace Yoke Seal</li></ol>

**Precision Medical, Inc.**  
300 Held Drive • Northampton, PA 18067  
800-272-7285 • 610-262-6090 • fax: 610-262-6080

504345rev3 25M 9/07 Printed in USA

## Setting up your **EASYPULSE5** *Oxygen Conserving Regulator*



**PRECISION**  **MEDICAL**®



Quick Start Guide only  
Refer to EasyPulse5 User Manual for complete operating instructions, warnings and cautions.

## EasyPulse5 Oxygen Conserving Regulator (Conservor)

# PARTS



# STEP 1

Inspect Conservor and Post Valve for damage or contaminants. Do NOT use if any are found. "Crack" cylinder valve by opening the valve slightly and closing.



# STEP 2

Ensure Conservor is in the OFF position and Yoke Seal is in place. Align Index Pins with Index Pin Alignment Holes in Post Valve. Hand tighten T-Handle. Do NOT use tools.



# STEP 3

When using a Precision Medical Carry Bag, slide the cylinder into the bag with the gauge facing the mesh. Attach the cannula to the Conservor's Cannula Connection.



# STEP 4

Slowly open the cylinder valve. Turn Dial to the prescribed setting. Zip the bag closed.



# STEP 5

Place cannula in nostrils and breathe through nose to begin Oxygen flow. If using a bag adjust the straps for maximum comfort.



# STEP 6

When finished using: Turn the cylinder valve to the OFF position. Turn Conservor Dial to "2 LPM Continuous" position. Wait for Oxygen to stop flowing from Conservor. Remove Conservor from Post Valve. Store Conservor and Carry Bag in a safe place.